

Maquet Servo-U Ventilator



Screen is a 'touch' screen, no dials or buttons to push



Alarm Silence for 2 Minutes



STANDBY
Press prior to disconnection to reduce gas flow and pause ventilation


Press & Hold 100 O₂ Boost, this will deliver 100% FiO₂ for 1 minute



Patient Data

Both patient and ventilator areas expandable by pressing the arrows


Ventilator Settings

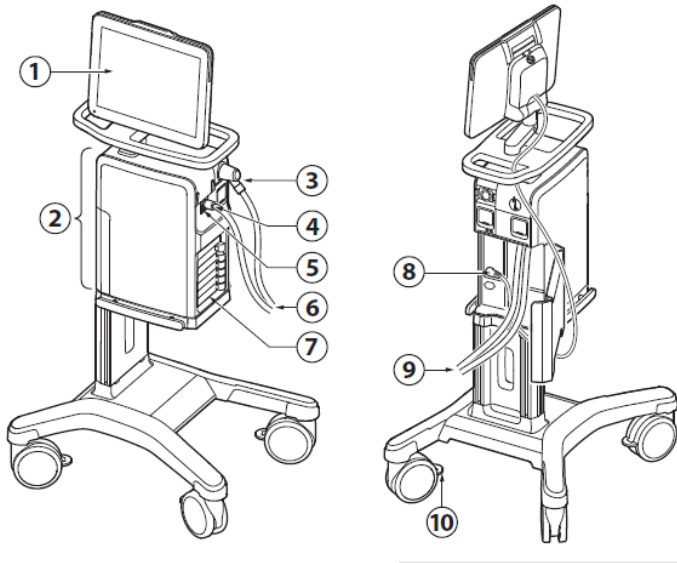

1. User interface
2. Patient unit
3. Expiratory inlet
4. Inspiratory outlet
5. Emergency air intake
6. Patient circuit
7. Module compartment
8. User interface control cable
9. Air and O₂ supply
10. Wheel lock



Maneuvers

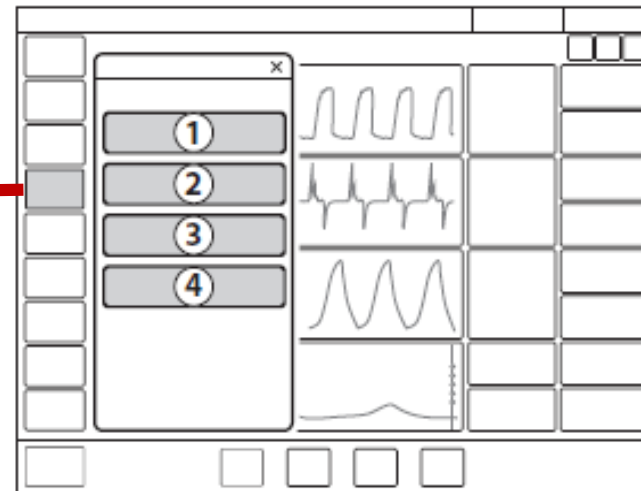

1. Manual breath
2. Static measurements
3. Nebulization
4. O₂ boost level

Here you will find the inspiratory and expiratory pause. Can measure the compliance (CSTAT), static resistance (RSTAT), plateau pressure (PPL)