


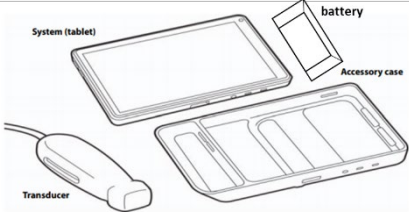


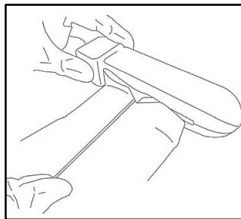

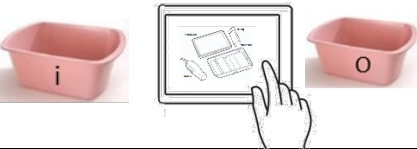





PREPPING ULTRASOUND FOR ENTERING PATIENT ROOM	
	<ul style="list-style-type: none"> <li>-Remove the transducer, battery, and tablet from grey cloth carry case. Leave carry case outside of the room.</li> <li>-Ensure that a spare battery, gel, and all required transducers, as well as wipes and all necessary procedural equipment are ready.</li> <li>-Ensure a basin or tray is available for passing out the device</li> </ul>
	Perform precautionary wipe of all surfaces with GREEN TOP wipe
CLEANING ULTRASOUND FOR PASSING OUT OF THE ROOM <u>BEFORE REMOVING PPE</u>	
	<p>Visually inspect the machine for any bodily fluid or tissue</p> <ul style="list-style-type: none"> <li>- If present, remove with wet gauze or wipes</li> </ul>
	Disconnect the probe and battery from tablet and remove the tablet from the accessory case to enable cleaning of all surfaces
	Sanitize Gloves with Alcohol Gel
	<ul style="list-style-type: none"> <li>-For each component, use a green-top wipe to wet all surfaces of ultrasound, wipe container, and gel bottle and keep wet for 1 minute (3 minutes if using blue-top wipe), then place component inside of pink basin (do not touch the inside of the basin).</li> <li>-Ensure that the slot in the transducer is cleaned by passing a wipe through from one side to another.</li> <li>-A cotton swab or other tool may help</li> </ul>
	
PASSING ULTRASOUND OUT OF THE ROOM AND SECOND CLEANING	
	<ul style="list-style-type: none"> <li>- Out-of-room provider dons full PPE, including mask and eye protection and obtains an additional pink basin (the outside basin)</li> <li>-In-room provider brings inside basin with clean components to door.</li> <li>-If 2<sup>nd</sup> provider not available, in room provider becomes out of room provider after doffing. Place pink basin on floor during transition</li> </ul>
	<p>Being careful not to contaminate gloves or outside basin, out-of-room provider opens door and transfers ultrasound components, one at a time, from inside basin into outside basin.</p> <p>-In-room provider should just hold basin.</p>
	<ul style="list-style-type: none"> <li>-Using a green-top wipe, wet all surfaces of probe, tablet, and case.</li> <li>-Also wipe gel bottle, wipe container, and outside basin</li> </ul>
	<p>Maintain wet for <b>1 minute</b> before device is decontaminated</p> <p><b>If blue wipes used, wet time for all process above must be <u>3 minutes</u></b></p>
	Wipe any residue from the tablet screen, and reassemble accessory case and battery
	Dispose of PPE in appropriate container