

PREPPING LUCAS FOR ENTERING PATIENT ROOM



Remove Device from carrying case and ensure that the necessary parts are ready:

Battery	Back Plate
Upper part	Stabilization Straps
Suction Cup	Cleaning wipe canister

CLEANING LUCAS DEVICE FOR PASSING OUT OF THE ROOM BEFORE REMOVING PPE



Dispose of suction cup in red bin



Wipe all surfaces of the device and straps with a wet cloth or bleach wipe in order to remove any body fluid or tissue.

- Separate Battery from Upper Part
- Separate Upper part from back board



Sanitize Gloves with Alcohol Gel



Wipe all surfaces with a **blue top wipe** and keep visibly wet for **3 minutes**, Recommended cleaning schedule:

NOTE: green top wipe will damage LUCAS—do not use.

3 min	<input type="checkbox"/>	Upper part + Battery
3 min	<input type="checkbox"/>	Back board
3 min	<input type="checkbox"/>	Straps and wipes can

As each part is cleaned, set it by the door

PASSING DEVICE OUT OF THE ROOM AND SECOND CLEANING



- Out of room provider dons gloves, gown, mask, and eye protection.
- Place a clean towel on floor.

While outside of room provider opens door, in-room provider carefully hands LUCAS components out. Outside provider places components on towel.



Outside provider wipes all surfaces with a **blue top wipe** and keep visibly wet for **3 minutes**, Recommended cleaning schedule:

NOTE: green top wipe will damage LUCAS—do not use.

3 min	<input type="checkbox"/>	Upper part + Battery
3 min	<input type="checkbox"/>	Back board
3 min	<input type="checkbox"/>	Straps and wipes can



Maintain wet for **3 minutes** before considering the device decontaminated
Allow to Air dry



Dispose of PPE in appropriate container