

**PRIOR TO ENTERING ROOM FOR ULTRASOUND EXAM OR PROCEDURE**



Ensure that all unnecessary materials are removed from the basket. Ensure that you have 1 bottle of ultrasound gel with you,



Perform precautionary wipe of all surfaces with GREEN TOP wipe  
**\*\*These surfaces need to stay wet for 2 min\*\***

**INSIDE THE ROOM, ONCE ULTRASOUND EXAM OR PROCEDURE IS COMPLETED, AFTER SANITIZING GLOVES WITH ALCOHOL GEL, BUT BEFORE REMOVING PPE**



Visually inspect the machine for any bodily fluid or tissue  
 - If present, remove with wet gauze or wipes



Using a green-top wipe, wet all surfaces of the machine including:

- the cords
- the keyboard
- the screen
- probes and cords
- the power cord
- the lid
- the wells
- gel bottle
- wipes containers

**\*\*These surfaces need to stay wet for 2 min\*\***



While still in PPE, move the wet machine out of the room  
**\*\*If another person is unavailable to help, use your foot to push the machine out\*\***

**OUTSIDE OF THE ROOM, WHILE WEARING CLEAN PPE**



Visually inspect the machine for any bodily fluid or tissue.  
**If present, remove with wipes**



Using a green-top wipe, wet all surfaces of the machine as above.



Maintain wet for **2 minutes** before considering the device decontaminated

